

PRESSALIT.

Product	PLUS grab bar 31.5"	
Item number	RT103112	
	VVS:777565412	
	NRF:6311211	
	RSK:8980551	
	EAN:5708590318684	
	HMI: 19084	
	NBS:N13:436	
Color	Anthracite gray	
Product description	PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroo A reliable solution with many different options, which can easily lay the grounds for increased independence. PLUS grab bar, diameter 1.3", 31.5", for fixed mounting. Materials: Powder-coated aluminium and plastic.	
Product capacity	Excl. mounting kit. Maximum load: 440 lbs. When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs. Grab bars are tested with a 1.5 safety factor in accordance with the requirements in	
Materials	ISO 17966:2016. Core material: Aluminium Surface material: Polyester lacquer Cover cap: Cycoloy plastic	
Ergonomy	The 1.3" diameter enables the rail to be held comfortably and firmly. The projection of 2" from the wall makes it easy to take hold of the rail.	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	



PRESSALIT.

Product	PLUS grab bar 31.5"
Item number	RT103112
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolared and stained, and the
	after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.
Declaration of conformity	This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.



PRESSALIT®

Product	


PLUS grab bar 31.5"

Item number

RT103112

Mounting in different wall types


Concrete >B25 V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
Solid Brick >= Mz12 V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
Solid Sand Lime Brick >= HIz12 V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
Aerated Concrete P4 V8666 - Mounting kit, 8 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.	

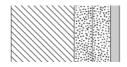

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood $\,$

V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8. NB! Plugs are not needed.

To be ordered separately.

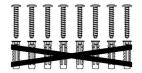



PRESSALIT®

Product	PLUS grab bar 31.5"	
Item number	RT103112	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	\triangle

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com