




PRESSALIT®

| | |
|----------------------------|---|
| Product | PLUS backrest, fixed height |
| Item number | R6515000 |
| Colour | <p>VVS:777538300 NRF:6190617 EAN:5708590325286 NBS:N13:426</p> <p>White</p> |
| Product description |  <p>PLUS backrest, 385 mm, soft feel, for SELECT toilet lifter. Materials: Aluminium, polyurethane foam with a plywood core. For mounting directly on SELECT toilet lifter, fixing kit is included.</p> <p>PLUS backrest, 385 mm, soft feel, for SELECT toilet lifter. Materials: Aluminium, polyurethane foam with a plywood core. For mounting directly on SELECT toilet lifter, fixing kit is included.</p> |
| Adjustability | Fixed mounting |
| Product capacity | 100 kg. |
| Materials | The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer. |
| Ergonomy | Soft and slip-resistant material that has a warm feel. |
| Warranty | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date. |

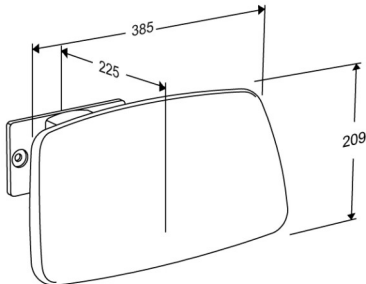

| | |
|--------------------|--|
| Product | PLUS backrest, fixed height |
| Item number | R6515000 |
| Cleaning | <p>PLUS backrest.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p>NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> |