

Product	MATRIX supports for wash basin, fixed height		
Item number	R2010182		
	VVS:636858800		
	NRF:6132553		
	EAN:5708590329338		
	HMI: 106241		
	NBS:N13:30,335		
Colour	Pale grey		
Product description	MATRIX supports are the low cost solution for MATRIX MEDIUM and ANGLE basins. They are available in a fixed height version and a manually height adjustable which is suitable for infrequent adjustment.		
	Supports for fixed height installation, pair of. Suitable for MEDIUM and ANGLE was basins with integrated handrails. Excl. mounting kit.		
Adjustability	Fixed mounting		
Product capacity	Maximum user weight (MUM): 160 kg. Maximum load (SWL): 150 kg. Suitable for wash basins weighing no more than 30 kg. When the wash basin is loaded with 150 kg, each wall fastener will be carrying 75 kg Wash basin brackets need to be able to carry the weight of the wash basin, as well a support the weight of the person who is leaning against the wash basin or who is using the wash basin to rise from seated position. Neither wash basin nor wash basin bracket are intended to carry the full body weight of a person , and the test methods applicable to wash basin brackets reflect this: ISO 17966 prescribes a static test on the front edge of the wash basin at 75% o maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 150 kg and the wash basin weighs 30 kg, it is – according to ISO 17966 – the equivalent of a user weighing 160 kg leaning on the front edge of the wash basin.		
Materials	Powder coated stainless steel		
I	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.		



Product	MATRIX supports for wash basin, fixed height	
Item number	R2010182	
Cleaning	VALUE & MATRIX wash basin brackets. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.	
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.	
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Klorilli	
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200	
	The product must be wiped down with a soft rag wrung out in clear water immediat after cleaning, as some materials may otherwise be discolored and stained, and th surface coating may deteriorate.	
Declaration of conformity	CE UK MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices. For the UK: - Medical Device Regulation (MDR) 2002 on medical devices.	



Product

Item number

MATRIX supports for wash basin, fixed height R2010182

Installation in different wall types

$\begin{array}{c} a \\ a $	Concrete >B25 V8600 - Mounting kit, 6 coach screws Ø6 x 70 - plugs SX8. To be ordered separately.	
	Solid Brick >= Mz12 V8600 - Mounting kit, 6 coach screws Ø6 x 70 - plugs SX8. To be ordered separately.	
	Solid Sand Lime Brick >= HIz12 V8600 - Mounting kit, 6 coach screws Ø6 x 70 - plugs SX8. To be ordered separately.	
	Aerated Concrete P4 It is not recommended to install this product in aerated concrete.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8600 - Mounting kit, 6 coach screws Ø6 x 70 - plugs SX8 NB! Plugs are not needed. To be ordered separately.	



Product Item number	MATRIX supports for wash basin, fixed height R2010182	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>.</u>
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com

