



<b>Product</b>	PLUS support arm, fixed height	
<b>Item number</b>	<p>R3070000  VVS:615851000  NRF:6190548  EAN:5708590337920  HMI: 101020  FDA: ILS/D135866</p>	
<b>Colour</b>	White	
<b>Product description</b>	<p>PLUS support arm, 700 mm, ergonomically shaped to provide a clear sense of where the support arm ends. For fixed mounting. Foldable.</p> <p>Materials: The arms are constructed of 10-micrometre anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core  Incl. fixing kit for brick wall.</p>	
<b>Adjustability</b>	Fixed mounting	
<b>Adjustment height</b>		
<b>Product capacity</b>	<p>Maximum load: 150 kg.  When the support arm is loaded with 150 kg, each wall fastener is loaded with 161 kg.</p> <p><i>PLUS support arms have been tested statically with 700 kg (close to the wall), 425 kg (in the middle) and 225 kg (the outermost point).</i></p>	
<b>Materials</b>	<p>The arms are primarily constructed in 10-micrometre anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core.</p> <p>The hinge arm is also made of soft SEBS.  Internal components and fasteners are primarily made of stainless steel.  The hinge barrel is capped with ABS polymer.</p>	
<b>Ergonomy</b>	<p>The oval shape gives an easy and comfortable surface to lean and push against.</p> <p>The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends.</p> <p>Soft and slip-resistant material that has a warm feel.</p>	
<b>Features</b>	<p>Folds against the wall.  The underside is shaped for the mounting of various accessories.</p>	





<b>Product</b>	PLUS support arm, fixed height
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
<b>Cleaning</b>	<p>PLUS support arms. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>
<b>Declaration of conformity</b>	<p>This product complies with the requirements of Council Directive 93/42/EEC concerning medical devices, as last amended by Council Directive 2007/47/EC of 5 September 2007. It also complies with all relevant requirements in the Council Directive 2006/42/EC (of 17 May 2006) on machinery.</p> <p><b>Intended use</b> The support arm is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.</p> <p><b>Operating environment</b> For indoor use in a wet environment, such as a bathroom or a washroom, in private homes, nursing homes and hospitals. For institutional use, the expected life time is 10 years. <i>Life cycle test performed according to ISO 17966:2016.</i></p> <p>For maintenance, please refer to user guide, which is enclosed with the product, and which can also be found on the website under "Downloads".</p>





# PRESSALIT®

<b>Product</b>	PLUS support arm, fixed height
----------------	--------------------------------



[www.pressalit.com](http://www.pressalit.com)

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®