




PRESSALIT®

Product Item number	VALUE support arm, 850 mm, with control panel R1083000
Colour	VVS:615892302 NRF:6191055 EAN:5708590346854 HMI: 138195 White
Product description	 <p>VALUE support arm, 850 mm, foldable, incl. operating panel for height adjustment of toilet lifter. Provides firm support when sitting, standing, or transferring to and from a toilet. To be mounted on the right (left hand operated). Height adjustable by 150 mm without using tools.. For fitting to toilet lifter (mounting profile included)</p> <p>VALUE support arm, incl. control panel for TL3/TL4 toilet lifter, 850 mm, foldable, height adjustable 150 mm without the use of tools (left hand operated). Provides firm support when sitting, standing, or transferring to and from a toilet. Materials: Rilsan-coated steel, diecast aluminium and end plug in nylon (PA). For fitting to TL3/TL4 electrical toilet lifter, ALBA wing mounting profile included. For mounting on user's right side.</p>
Adjustability	Height adjustable
Adjustment height	150 mm
Product capacity	Maximum load: 120 kg
Materials	The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The end caps are manufactured from nylon (PA). The ALBA wing mounting profile is made from diecast aluminium.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

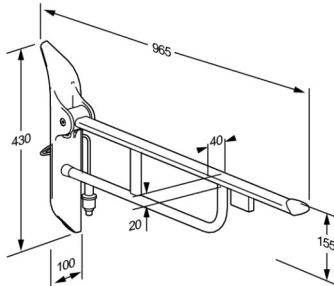

Product	VALUE support arm, 850 mm, with control panel
Item number	R1083000
Cleaning	<p>VALUE support arms.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS:</p> <p>Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean)</p> <p>NON-DISINFECTANTS:</p> <p>Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
Declaration of conformity	<p>CE MDR-RoHS This product is in compliance with all relevant requirements of:</p> <ul style="list-style-type: none">- Regulation (EU) 2017/745 on medical devices- Council Directive 2011/65/EC on RoHS.



PRESSALIT®

Product	VALUE support arm, 850 mm, with control panel
Item number	R1083000

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®