


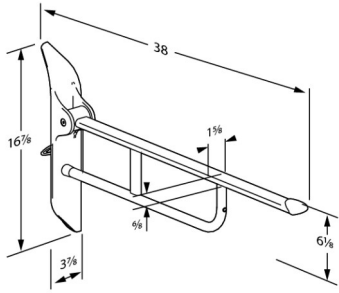

PRESSALIT®

<b>Product</b> <b>Item number</b>	VALUE fold down grab bar, 33.5" R1085000
<b>Color</b>	VVS:615892502 NRF:6191056 EAN:5708590346861 HMI: 138199  White
<b>Product description</b>	 <p>VALUE and VALUE II fold down grab bars with the ALBA wing mounting profile are specifically designed for use with SELECT TL3 and TL4 toilet lifters. The height adjustment of the toilet lifter is done only with the control panel on the fold down grab bar.</p> <p>VALUE fold down grab bar, 33.5", foldable, height adjustable 5.9" without the use of tools (left hand operated) . Provides firm support when sitting, standing, or transferring to and from a toilet. Materials: Rilsan-coated steel, diecast aluminum and end plug in nylon (PA). For fitting to TL3/TL4 manual toilet lifter, ALBA wing mounting profile included. For mounting on user's right side.</p>
<b>Adjustability</b>	Height adjustable
<b>Adjustment height</b>	5,91 inch
<b>Product capacity</b>	Maximum load: 265 lbs
<b>Materials</b>	The fold down grab bar is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The end caps are manufactured from nylon (PA). The mounting profile is made from diecast aluminum.
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



# PRESSALIT®

<b>Product</b>	VALUE fold down grab bar, 33.5"
<b>Item number</b>	R1085000
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean)</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
<b>Declaration of conformity</b>	<p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>



# PRESSALIT®

<b>Product</b>	VALUE fold down grab bar, 33.5"
<b>Item number</b>	R1085000

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®