

PRESSALIT.

Product	PLUS wall track, grooved, 8.3". For right hand operated fold down grab bar		
Item number	R9862112		
	VVS:615842102		
	EAN:5708590352299		
	1%		
Color	Anthracite gray		
Product description	PLUS wall track, grooved, for vertical mounting, 8.3". For height adjustable, right-hand operated fold down grab bars. Below a selection of products for fitting to the wall track. The cover plate hides bolts and wires and facilitates cleaning. Excl. mounting kit. See Technical datasheet (Downloads) for information on mounting kits suitable various wall types.		
Adjustability	Fixed mounting		
Materials	The wall track is primarily made of 10-micrometer anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polyn		
Ergonomy	The vertical track enables height adjustment of the different products. If combined with the horizontal track, the available space in the bathroom can be utilized optimally. In environments with multiple users, the vertical track makes it possible to adjust the different products to individual user heights.		
Features	The front cover plate conceals mounting bolts and any cables.		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.		



PRESSALIT.

Product Item number	PLUS wall track, grooved, 8.3". For right hand operated fold down grab bars R9862112
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: CaviCide The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.



PRESSALIT.

Product							
Item	numbe						

PLUS wall track, grooved, 8.3". For right hand operated fold down grab bars R9862112

Mounting in different wall types

9		
	Concrete >B25 V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Brick >= Mz12 V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Sand Lime Brick >= HIz12 V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	Aerated Concrete P4 V8653 - Mounting kit, 4 bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



PRESSALIT®

Product Item number	PLUS wall track, grooved, 8.3". For right hand operated fold down grab bars R9862112				
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		ÛP UP SARE		
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	81]	⇔ 5		
	Drywall (2 x 12,5 mm) water-resistant				
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	4	<u> </u>		

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com

PRESSALIT.