

## PRESSALIT.

Product	PLUS shower seat 310, fixed height		
Item number	R7304000		
	VVS:777513000		
	NRF:6317084		
	EAN:5708590359274		
	HMI: 121181 EDA: INIV/D135865		
	FDA: INN/D135865		
Colour	White		
	310 365		
Product description	PLUS shower seats are offered in a number of colours. They are available as both		
	fixed height, electrically adjustable and manually adjustable, and with an option of back and armrests.		
	They are suitable for wall-mounting or for fitting to Pressalit PLUS wall tracks.		
	PLUS shower seat 310. For fixed mounting. The seat is foldable.		
	Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminiun profile.		
	Excl. mounting kit.  Replaces R7310		
	Replaces K1310		
Adjustability	Fixed mounting		
Product capacity	Maximum user mass: 200 kg.		
	When the seat is loaded with 200 kg, each wall fastener is loaded with 113 kg.		
	The seat has been tested statically with safety factor 1.5, which is a requirement		
	according to the applied assistive products standards.		
Materials	The shower seat is made of polyurethane (PU) foam with a plywood core and this is		
	covered with an in-mould coated surface that is protected using a clear PU lacquer. Internal components and fasteners are primarily made of stainless steel. The hinge		
	barrel is capped with ABS.		
Ergonomy	Ergonomically designed seat - contact surface are 450 x 437 mm. Soft and slip-resistant material that has a warm feel.		
Features	The seat can fold up when not in use or to maximise floor space.		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.		
I			



# PRESSALIT.

Product Item number	PLUS shower seat 310, fixed height R7304000	
Cleaning	PLUS shower seat. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.	
	Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.	
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.	
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.	
Declaration of conformity	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.	



### PRESSALIT.

PLUS shower seat 310, fixed height

Item number

R7304000

### Installation in different wall types

'S. \\ \'\'\'\'\'\\\\\\\\\\\\\\\\\\\\\\\
·
D. C C. a. ∇ . L
D. D. D. Q V
~ . D . G

#### Concrete >B25

V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10.

To be ordered separately.






#### Solid Brick >= Mz12

V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10.

To be ordered separately.



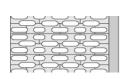

#### Solid Sand Lime Brick >= Hlz12

V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs

To be ordered separately.







#### **Aerated Concrete P4**

V8654 - Mounting kit, 4 countersunk bolts M8 x 35, plugs FTP M 8.

To be ordered separately.

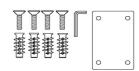


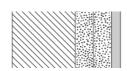

### Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. NB! Plugs are not needed.

To be ordered separately.

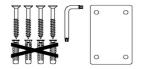



## PRESSALIT®

Product Item number	PLUS shower seat 310, fixed height R7304000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood  Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)  Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	!

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com

