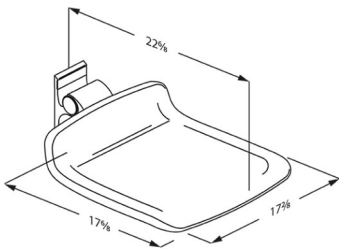




# PRESSALIT®

<b>Product</b> <b>Item number</b>	PLUS shower seat 450, fixed height R7404035
<b>Color</b>	<p>NRF:6317102 EAN:5708590359489 HMI: 124480 FDA:INN/D135865</p> <p>Red (RAL 3003)</p> 
<b>Product description</b>	<p>PLUS shower seats are offered in a number of colours. They are available as both fixed height, electrically adjustable and manually adjustable, with and without aperture, and with an option of back and armrests. They are suitable for wall-mounting or for fitting to Pressalit PLUS wall tracks.</p> <p>PLUS shower seat 450. For fixed mounting. The seat is foldable. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile. Excl. mounting kit.</p>
<b>Adjustability</b> <b>Product capacity</b>	<p>Fixed mounting</p> <p>Maximum user mass: 440 lbs. When the seat is loaded with 440 lbs, each wall fastener is loaded with 187 lbs.</p> <p><i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i></p>
<b>Materials</b> <b>Ergonomy</b> <b>Features</b>	<p>The shower seat is made of polyurethane (PU) foam with a plywood core and this is covered with a moulded coated surface that is protected using a clear PU lacquer. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS.</p> <p>Ergonomically designed seat - contact surface are 17.7" x 17.2". Soft and slip-resistant material that has a warm feel.</p> <p>The seat can fold up when not in use or to maximize floor space.</p>
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



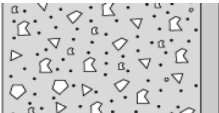
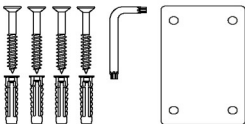

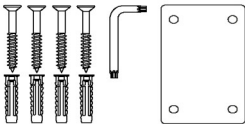
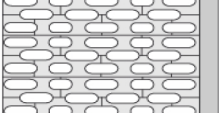
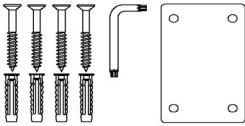
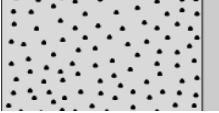
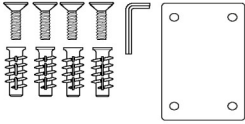
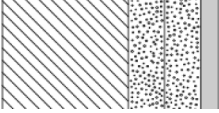
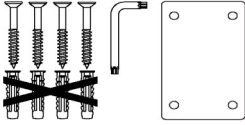
<b>Product</b>	PLUS shower seat 450, fixed height
<b>Item number</b>	R7404035
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p><b>NON-DISINFECTANTS:</b> Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
<b>Declaration of conformity</b>	<p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>



PRESSALIT®

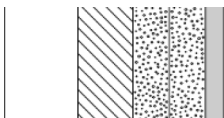
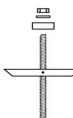

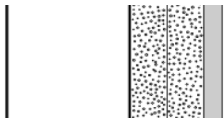
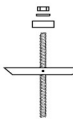

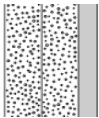

<b>Product</b>	PLUS shower seat 450, fixed height
<b>Item number</b>	R7404035

## Mounting in different wall types

	<p><b>Concrete &gt;B25</b></p> <p>V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. To be ordered separately.</p> 
	<p><b>Solid Brick &gt;= Mz12</b></p> <p>V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. To be ordered separately.</p> 
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. To be ordered separately.</p> 
	<p><b>Aerated Concrete P4</b></p> <p>V8654 - Mounting kit, 4 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.</p> 
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.</p> 



# PRESSALIT®

<b>Product</b> <b>Item number</b>	PLUS shower seat 450, fixed height R7404035		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b> Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.		

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

[pressalit.com](http://pressalit.com)

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®