



<b>Product</b> <b>Item number</b>	PLUS shower seat 450 with aperture, manually height adjustable R7415000
<b>Colour</b>	VVS:777515200 NRF:6317112 EAN:5708590359588 HMI: 124332 FDA:INN/D135865 White
<b>Product description</b>	<div data-bbox="1018 488 1497 824" data-label="Image"> </div> <p>PLUS shower seats are offered in a number of colours. They are available as both fixed height, electrically adjustable and manually adjustable, with and without aperture, and with an option of back and armrests. They are suitable for wall-mounting or for fitting to Pressalit PLUS wall tracks.</p> <p>PLUS shower seat 450 with aperture. Manually height adjustable by 240 mm, without the use of tools. The seat is foldable.          Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile.          Excl. mounting kit.  <i>Replaces R7411</i></p>
<b>Adjustability</b> <b>Adjustment height</b> <b>Product capacity</b>	Height adjustable 240 mm Maximum user mass: 200 kg. When the seat is loaded with 200 kg, each wall fastener is loaded with 47 kg. <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
<b>Materials</b> <b>Ergonomy</b> <b>Features</b>	<p>The shower seat is made of polyurethane (PU) foam with a plywood core and this is covered with an in-mould coated surface that is protected using a clear PU lacquer. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS.</p> <p>Ergonomically designed seat - contact surface are 450 x 437 mm. Soft and slip-resistant material that has a warm feel.</p> <p>The seat can fold up when not in use or to maximise floor space.</p>



# PRESSALIT®

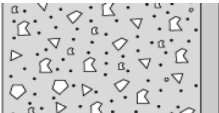
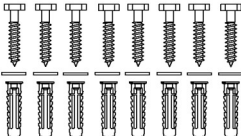

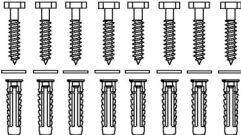
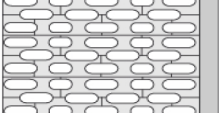
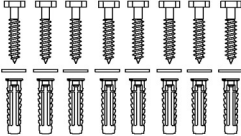
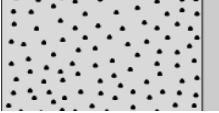
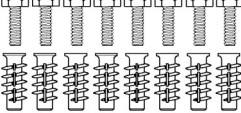
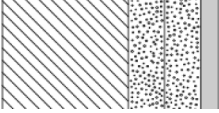
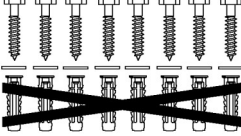
<b>Product</b>	PLUS shower seat 450 with aperture, manually height adjustable
<b>Item number</b>	R7415000
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
<b>Cleaning</b>	<p>PLUS shower seat.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p><b>NON-DISINFECTANTS:</b> Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>
<b>Declaration of conformity</b>	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.



PRESSALIT®

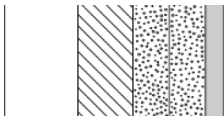
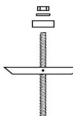

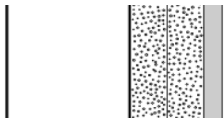
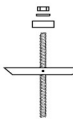

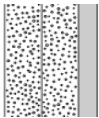
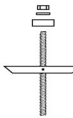

<b>Product</b>	PLUS shower seat 450 with aperture, manually height adjustable
<b>Item number</b>	R7415000

## Installation in different wall types

	<p><b>Concrete &gt;B25</b></p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.</p> 
	<p><b>Solid Brick &gt;= Mz12</b></p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.</p> 
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.</p> 
	<p><b>Aerated Concrete P4</b></p> <p>V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.</p> 
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.</p> 



# PRESSALIT®

Product Item number	PLUS shower seat 450 with aperture, manually height adjustable R7415000		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</b> Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.		

pressalit.com