


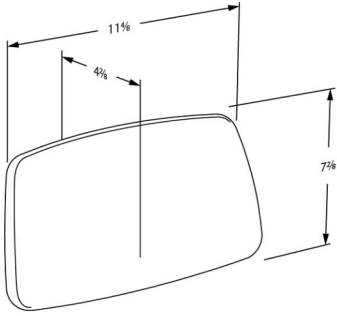

<b>Product</b> <b>Item number</b>	PLUS backrest, fixed height R6304000
<b>Color</b>	VVS:777538100 NRF:6317025 EAN:5708590365282 HMI: 124221  White
<b>Product description</b>	 <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable.          They are suitable for wall-mounting behind a toilet without cistern or mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 11.5" x 7.5", soft feel.          Fixed height, for mounting directly on the wall behind a wall-mounted toilet without cistern.          Materials: Aluminium and polyurethane foam with a plywood core.          Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p>
<b>Adjustability</b> <b>Product capacity</b>	Fixed mounting  Maximum load: 110 lbs. Maximum user weight: 220 lbs. When the backrest is loaded with 110 lbs, each wall fastener will be carrying 130 lbs. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i> <b><i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 110 lbs, it is – according to ISO 17966 – the equivalent of a user weighing 220 lbs leaning on the backrest.</i></b>
<b>Materials</b>	The backrest is made of polyurethane (PU) foam which is covered with a moulded coated surface that is protected using a clear PU lacquer.
<b>Ergonomy</b>	Soft and slip-resistant material that has a warm feel.
<b>Features</b>	For toilet without cistern
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



# PRESSALIT®

<b>Product</b>	PLUS backrest, fixed height
<b>Item number</b>	R6304000
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> CaviCide</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
<b>Declaration of conformity</b>	<p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>

[pressalit.com](http://pressalit.com)

## PRESSALIT®