




PRESSALIT®

| | |
|--|---|
| Product Item number | PLUS backrest, fixed height R6404277 |
| Colour | NRF:6317035 EAN:5708590365404 HMI: 124215 Lime green |
| Product description |  <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or for mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 385 x 209 mm, soft feel. Fixed height, for mounting directly on the wall behind a wall-mounted elongated toilet without cistern. Materials: Aluminium and polyurethane foam with a plywood core. Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p> |
| Adjustability Product capacity | Fixed mounting Maximum load: 50 kg. Maximum user weight: 100 kg. When the backrest is loaded with 50 kg, each wall fastener will be carrying 118 kg. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i> <i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 50 kg, it is – according to ISO 17966 – the equivalent of a user weighing 100 kg leaning on the backrest.</i> |
| Materials Ergonomy Features | The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer. Soft and slip-resistant material that has a warm feel. For toilet without cistern |
| Warranty | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date. |

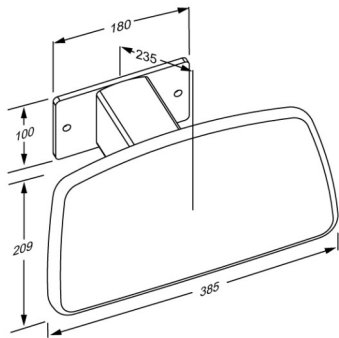

| | |
|----------------------------------|---|
| Product | PLUS backrest, fixed height |
| Item number | R6404277 |
| Cleaning | <p>PLUS backrest.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS:</p> <p>Erisan OXY+</p> <p>Tvättspit 70%</p> <p>CaviCide surface disinfectant decontaminant cleaner</p> <p>Rodalon Desinfektionsspray</p> <p>Terralin protect (SCHÜLKE+)</p> <p>Dismonzon Plus (BODE Chemie GmbH)</p> <p>NON-DISINFECTANTS:</p> <p>Wetrok Caledor (Wetrok)</p> <p>Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS:</p> <p>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> <p>CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.</p> <p>Expected service life: 10 years.</p> <p>Contraindications for use: It is not recommended that people who need to lie down while being changed use the backrest for support.</p> |
| Declaration of conformity | |



PRESSALIT®

| | |
|--------------------|-----------------------------|
| Product | PLUS backrest, fixed height |
| Item number | R6404277 |

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®