




PRESSALIT®

Product Item number	PLUS backrest, fixed height, for toilet lifter R6408112
Colour	VVS:777530312 NRF:6317037 EAN:5708590365428 HMI: 124218 Anthracite grey
Product description	 <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or for mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 385 x 209 mm, soft feel. Fixed height, for mounting directly on SELECT TL1, TL2, TL3 or TL4 toilet lifter. Materials: Aluminium and polyurethane foam with a plywood core. Mounting kit (for mounting on toilet lifter) included.</p>
Adjustability Product capacity	Fixed mounting Maximum load: 50 kg. Maximum user weight: 100 kg. When the backrest is loaded with 50 kg, each wall fastener will be carrying 118 kg. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i> <i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 50 kg, it is – according to ISO 17966 – the equivalent of a user weighing 100 kg leaning on the backrest.</i>
Materials	The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer.
Ergonomy	Soft and slip-resistant material that has a warm feel.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

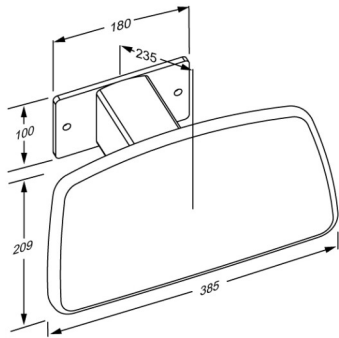

Product	PLUS backrest, fixed height, for toilet lifter
Item number	R6408112
Cleaning	<p>PLUS backrest.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS:</p> <p>Erisan OXY+</p> <p>Tvättspit 70%</p> <p>CaviCide surface disinfectant decontaminant cleaner</p> <p>Rodalon Desinfektionsspray</p> <p>Terralin protect (SCHÜLKE+)</p> <p>Dismonzon Plus (BODE Chemie GmbH)</p> <p>NON-DISINFECTANTS:</p> <p>Wetrok Caledor (Wetrok)</p> <p>Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS:</p> <p>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> <p>CE UK MDR This product is in compliance with all relevant requirements of:</p> <p>- Regulation (EU) 2017/745 on medical devices.</p> <p>For the UK: - Medical Device Regulation (MDR) 2002 on medical devices.</p> <p>Expected service life: 10 years.</p> <p>Contraindications for use: It is not recommended that people who need to lie down while being changed use the backrest for support.</p>
Declaration of conformity	



PRESSALIT®

Product	PLUS backrest, fixed height, for toilet lifter
Item number	R6408112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®