


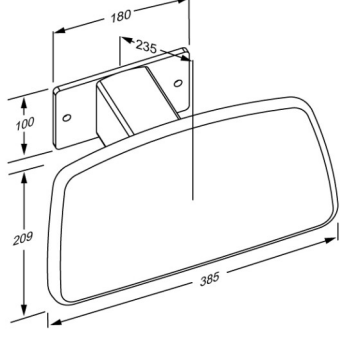

<p>Product Item number</p>	<p>PLUS backrest, fixed height, for toilet lifter R6408112</p>
<p>Colour Product description</p>	<p>VVS:777530312 NRF:6317037 EAN:5708590365428 HMI: 124218</p> <p>Anthracite grey</p>  <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or for mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 385 x 209 mm, soft feel. Fixed height, for mounting directly on SELECT TL1, TL2, TL3 or TL4 toilet lifter. Materials: Aluminium and polyurethane foam with a plywood core. Mounting kit (for mounting on toilet lifter) included.</p>
<p>Adjustability Product capacity</p>	<p>Fixed mounting</p> <p>Maximum load: 50 kg. Maximum user weight: 100 kg. When the backrest is loaded with 50 kg, each wall fastener will be carrying 118 kg. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i> <i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 50 kg, it is – according to ISO 17966 – the equivalent of a user weighing 100 kg leaning on the backrest.</i></p>
<p>Materials Ergonomy</p>	<p>The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer.</p> <p>Soft and slip-resistant material that has a warm feel.</p>
<p>Warranty</p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>



Product	PLUS backrest, fixed height, for toilet lifter
Item number	R6408112
Cleaning	<p>PLUS backrest. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>
Declaration of conformity	<p>CE UK MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices. For the UK: - Medical Device Regulation (MDR) 2002 on medical devices.</p> <p>Expected service life: 10 years. Contraindications for use: It is not recommended that people who need to lie down while being changed use the backrest for support.</p>