




PRESSALIT®

<b>Product</b> <b>Item number</b>	PLUS backrest, vertical and horizontal adjustable R6444000
<b>Color</b>	VVS:777530600 NRF:6317049 EAN:5708590365503 HMI: 124205  White
<b>Product description</b>	 <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 15.2" x 8.2", soft feel.          Height adjustable 150 mm and sideways adjustable, for mounting on a horizontal walltrack behind a wall-mounted toilet without cistern.          Can be removed easily when not needed and reattached again without the use of tools.          Materials: Aluminium and polyurethane foam with a plywood core.          For fitting to horizontal PLUS wall track (not included).</p>
<b>Adjustability</b> <b>Adjustment height</b>  <b>Product capacity</b>	Vertical and horizontal adjustable  5,91 inch  Maximum load: 110 lbs. Maximum user weight: 220 lbs. When the backrest is loaded with 110 lbs, each wall fastener will be carrying 73 lbs. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i> <b><i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 110 lbs, it is – according to ISO 17966 – the equivalent of a user weighing 220 lbs leaning on the backrest.</i></b>
<b>Materials</b>  <b>Ergonomy</b>	The backrest is made of polyurethane (PU) foam which is covered with a moulded coated surface that is protected using a clear PU lacquer.  Soft and slip-resistant material that has a warm feel.

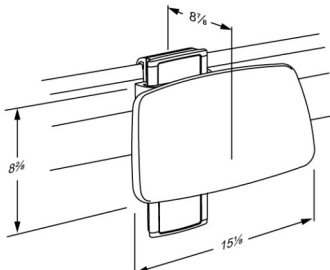

# PRESSALIT®

<b>Product</b>	PLUS backrest, vertical and horizontal adjustable
<b>Item number</b>	R6444000
<b>Features</b>	For tankless toilet
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p><b>NON-DISINFECTANTS:</b> Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
<b>Declaration of conformity</b>	<p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>



# PRESSALIT®

<b>Product</b>	PLUS backrest, vertical and horizontal adjustable
<b>Item number</b>	R6444000

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®