




PRESSALIT®

Product Item number	PLUS backrest, height and sideways adjustable R6444277
Colour	NRF:6317054 EAN:5708590365558 HMI: 124205 Lime green
Product description	 <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or for mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 385 x 209 mm, soft feel. Height adjustable 150 mm and sideways adjustable, for mounting on a horizontal walltrack behind a wall-mounted toilet without cistern. Can be removed easily when not needed and reattached again without the use of tools. Materials: Aluminium and polyurethane foam with a plywood core. For fitting to horizontal PLUS wall track (not included).</p>
Adjustability Adjustment height Product capacity	Height and sideways adjustable 150 mm Maximum load: 50 kg. Maximum user weight: 100 kg. When the backrest is loaded with 50 kg, each wall fastener will be carrying 33 kg. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i> <i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 50 kg, it is – according to ISO 17966 – the equivalent of a user weighing 100 kg leaning on the backrest.</i>
Materials Ergonomy	The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer. Soft and slip-resistant material that has a warm feel.

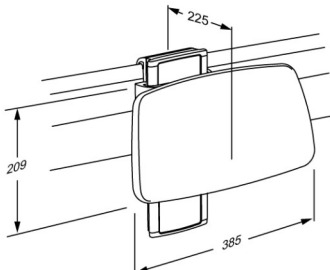

PRESSALIT®

Product	PLUS backrest, height and sideways adjustable
Item number	R6444277
Features	For toilet without cistern
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	<p>PLUS backrest.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Erisan OXY+ Tvättspit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p>NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>
Declaration of conformity	<p>CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.</p> <p>Expected service life: 10 years.</p> <p>Contraindications for use: It is not recommended that people who need to lie down while being changed use the backrest for support.</p>



PRESSALIT®

Product	PLUS backrest, height and sideways adjustable
Item number	R6444277

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®