




<p><b>Product</b> <b>Item number</b></p>	<p>PLUS backrest, removable R6409000</p>
<p><b>Colour</b>  <b>Product description</b></p>	<p>VVS:777530400 NRF:6317039 EAN:5708590366296 HMI: 124211</p> <p>White</p>  <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or for mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 385 x 209 mm, soft feel. Fixed height, for mounting directly on SELECT TL1, TL2, TL3 or TL4 toilet lifter. Can be removed easily when not needed and reattached again without the use of tools. Materials: Aluminium and polyurethane foam with a plywood core. Mounting kit (for mounting on toilet lifter) included.</p>
<p><b>Adjustability</b> <b>Product capacity</b></p>	<p>Fixed mounting</p> <p>Maximum load: 50 kg. Maximum user weight: 100 kg. When the backrest is loaded with 50 kg, each wall fastener will be carrying 118 kg. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i> <b><i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 50 kg, it is – according to ISO 17966 – the equivalent of a user weighing 100 kg leaning on the backrest.</i></b></p>
<p><b>Materials</b> <b>Ergonomy</b></p>	<p>The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer.</p> <p>Soft and slip-resistant material that has a warm feel.</p>
<p><b>Warranty</b></p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>

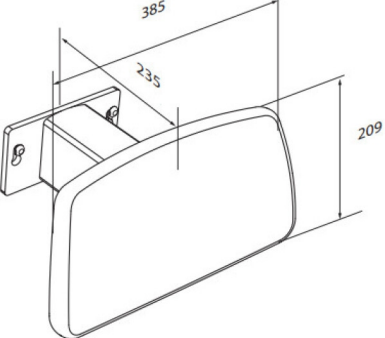

<b>Product</b>	PLUS backrest, removable
<b>Item number</b>	R6409000
<b>Cleaning</b>	<p>PLUS backrest. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p><b>NON-DISINFECTANTS:</b> Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>
<b>Declaration of conformity</b>	<p>CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.</p> <p><b>Expected service life:</b> 10 years. <b>Contraindications for use:</b> It is not recommended that people who need to lie down while being changed use the backrest for support.</p>



PRESSALIT®

---

<b>Product</b>	PLUS backrest, removable
<b>Item number</b>	R6409000

---

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

PRESSALIT®