

Product	PLUS support arm with integrated counter-balance, 850 mm	
Item number	R360085035	
	NRF:6191073	
	EAN:5708590367934	
	HMI: 123553	
	FDA: ILS/D135866	
Colour	Red (RAL 3003)	
Product description	All PLUS support arms come with an integrated counter-balance, which ensures a controlled movement between horizontal and vertical positions. Suitable for use with toilet pans projecting 700 mm or floor standing toilets. Replaces discontinued models R3085 and RF085.	
	PLUS support arm, counter-balanced. Projection 850 mm, ergonomically shaped to provide a clear sense of where the support arm ends. For fixed mounting. Foldable. Materials: The arms are constructed of 10-micrometre anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core Excl. mounting kit.	
Adjustability	Fixed mounting	
Product capacity	Maximum load: 150 kg. When the support arm is loaded with 150 kg, each wall fastener is loaded with 196 kg.	
	PLUS support arms have been tested statically with 700 kg (close to the wall), 425 k (in the middle) and 225 kg (the outermost point).	
Materials	The arms are primarily constructed in 10-micrometre anodized aluminium, with a nor slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core. The hinge arm is also made of soft SEBS.	
	Internal components and fasteners are primarily made of stainless steel.	



Product Item number	PLUS support arm with integrated counter-balance, 850 mm R360085035 The oval shape gives an easy and comfortable surface to lean and push against The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends. Soft and slip-resistant material that has a warm feel.	
Ergonomy		
Features	Folds against the wall. The underside is shaped for the mounting of various accessories.	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	



Product	PLUS support arm with integrated counter-balance, 850 mm R360085035	
ltem number		
Cleaning		
	PLUS support arms. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.	
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.	
	DISINFECTANTS:	
	Clinimax Difficil-S Erisan OXY+	
	Sprit (husholdningssprit)	
	Suma Bac D10	
	Virkon	
	Rodalon Desinfektionsspray	
	Lysoformin Spezial CaviCide	
	Sufanios Premium	
	Tvättsprit 70%	
	Oxyplus-acid 2% (Diversey)	
	WipeClean UniversalCleaning	
	Chrolorine Disinfection Wipes	
	NON-DISINFECTANTS:	
	Suma Star free D1	
	Taski 100	
	Taski 200	
	Milizid Citro	
	The product must be wiped down with a soft rag wrung out in clear water immedia after cleaning, as some materials may otherwise be discolored and stained, and t surface coating may deteriorate.	
	DECALCIFICATION OF ALUMINUM PARTS:	
	Pressalit recommends that chalky deposits be removed from wall tracks and othe aluminum parts using a solution of three parts 30% acetic acid to seven parts tap	

Declaration of conformity

CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.

water. When the aluminum parts have been washed down with this solution, the parts

must be wiped over with a damp cloth wrung out in clear water.



Product Item number	PLUS support arm with integrated counter-balance, 850 mm R360085035
	Intended use The support arm is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.
	Operating environment For indoor use in a wet environment, such as a bathroom or a washroom, in private
	homes, nursing homes and hospitals. For institutional use, the expected life time is 10 years. Life cycle test performed according to ISO 17966:2016.
	For maintenance, please refer to user guide, which is enclosed with the product, and which can also be found on the website under "Downloads".



Item number

PLUS support arm with integrated counter-balance, 850 mm

R360085035


Installation in different wall types

In. V ((
.C	
D . 7 a	
0.0.0.0	
Δ. D. G	

Concrete >B25

V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10.

To be ordered separately.

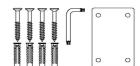




Solid Brick >= Mz12

V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10.

To be ordered separately.




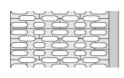

Solid Sand Lime Brick >= Hlz12

V8604 - Mounting kit, 4 countersunk screws \emptyset 8 x 70, plugs SX10.

To be ordered separately.

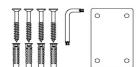




Aerated Concrete P4

V8654 - Mounting kit, 4 countersunk bolts M8 x 35, plugs FTP M 8.

To be ordered separately.

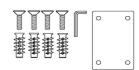




Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. NB! Plugs are not needed.

To be ordered separately.

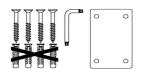



PRESSALIT®

Product Item number	PLUS support arm with integrated counter-balance, 850 R360085035	mm
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>^!</u>

pressalit.com