


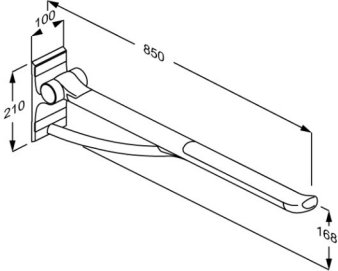

| | |
|--|---|
| <p>Product Item number</p> | <p>PLUS support arm with integrated counter-balance, 850 mm R360085277</p> |
| <p>Colour Product description</p> | <p>NRF:6191076 EAN:5708590367965 HMI: 123553 FDA: ILS/D135866</p>  <p>Lime green</p> <p>All PLUS support arms come with an integrated counter-balance, which ensures a controlled movement between horizontal and vertical positions. Suitable for use with toilet pans projecting 700 mm or floor standing toilets. <i>Replaces discontinued models R3085 and RF085.</i></p> <p>PLUS support arm, counter-balanced. Projection 850 mm, ergonomically shaped to provide a clear sense of where the support arm ends. For fixed mounting. Foldable. Materials: The arms are constructed of 10-micrometre anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core Excl. mounting kit.</p> |
| <p>Adjustability Product capacity</p> | <p>Fixed mounting</p> <p>Maximum load: 150 kg. When the support arm is loaded with 150 kg, each wall fastener is loaded with 196 kg.</p> <p><i>PLUS support arms have been tested statically with 700 kg (close to the wall), 425 kg (in the middle) and 225 kg (the outermost point).</i></p> |
| <p>Materials</p> | <p>The arms are primarily constructed in 10-micrometre anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core. The hinge arm is also made of soft SEBS. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS polymer.</p> |



PRESSALIT®

| | |
|--------------------------------------|---|
| Product Item number | PLUS support arm with integrated counter-balance, 850 mm R360085277 |
| Ergonomy | <p>The oval shape gives an easy and comfortable surface to lean and push against.</p> <p>The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends.</p> <p>Soft and slip-resistant material that has a warm feel.</p> |
| Features | <p>Folds against the wall.</p> <p>The underside is shaped for the mounting of various accessories.</p> |
| Warranty | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date. |



| | |
|----------------------------------|---|
| Product | PLUS support arm with integrated counter-balance, 850 mm |
| Item number | R360085277 |
| Cleaning | <p>PLUS support arms. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Oxyplus-acid 2% (Diversey) WipeClean UniversalCleaning Chlorine Disinfection Wipes</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> |
| Declaration of conformity | CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices. |



PRESSALIT®

| | |
|--------------------------------------|--|
| Product Item number | PLUS support arm with integrated counter-balance, 850 mm R360085277 |
| | <p>Intended use The support arm is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.</p> <p>Operating environment For indoor use in a wet environment, such as a bathroom or a washroom, in private homes, nursing homes and hospitals. For institutional use, the expected life time is 10 years. <i>Life cycle test performed according to ISO 17966:2016.</i></p> <p>For maintenance, please refer to user guide, which is enclosed with the product, and which can also be found on the website under "Downloads".</p> |









| | |
|--------------------|--|
| Product | PLUS support arm with integrated counter-balance, 850 mm |
| Item number | R360085277 |

Installation in different wall types

| | | |
|--|--|--|
| | <p>Concrete >B25 V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. To be ordered separately.</p> | |
| | <p>Solid Brick >= Mz12 V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. To be ordered separately.</p> | |
| | <p>Solid Sand Lime Brick >= Hlz12 V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. To be ordered separately.</p> | |
| | <p>Aerated Concrete P4 V8654 - Mounting kit, 4 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.</p> | |
| | <p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.</p> | |

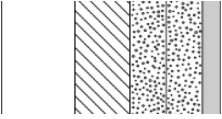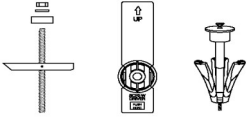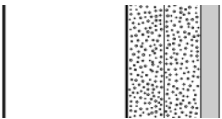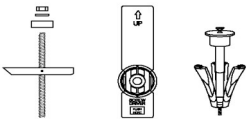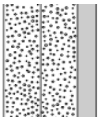

| | |
|---|---|
| Product Item number | PLUS support arm with integrated counter-balance, 850 mm R360085277 |
|  | Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.  |
|  | Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.  |
|  | Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.  |