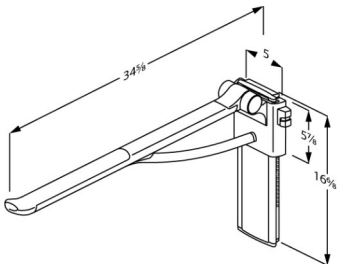




# PRESSALIT®

|   |   |
|---|---|
| <b>Product</b><br><b>Item number</b>  | PLUS fold down grab bar with integrated counter-balance, 33.5", right hand operated<br>R370485277   |
| <b>Color</b>  | NRF:6191099<br>EAN:5708590368269<br>HMI: 123500<br>FDA: ILS/D135866<br><br>Lime green   |
| <b>Product description</b>  |  <p>All PLUS fold down grab bars come with an integrated counter-balance, which ensures a controlled movement between horizontal and vertical positions. Suitable for use with elongated toilet pans or floor standing toilets.</p> <p>PLUS fold down grab bar, counter-balanced. Projection 34.7", ergonomically shaped to provide a clear sense of where the fold down grab bar ends. Right hand operated. Height adjustable by 9.5" without using tools. Detachable and foldable. Anthracite cover plate for vertical wall track.</p> <p>Materials: The arms are constructed of 10-micrometer anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core<br/>         Excl. mounting kit.</p> |
| <b>Adjustability</b><br><b>Adjustment height</b><br><br><b>Product capacity</b> | Height adjustable<br>9,46 inch<br><br>Maximum load: 330 lbs.<br>When the fold down grab bar is loaded with 330 lbs, each wall fastener in the vertical wall track is loaded with 110 lbs.<br><br><i>PLUS fold down grab bars have been tested statically with 1540 lbs (close to the wall), 935 lbs (in the middle) and 495 lbs (the outermost point).</i>  |
| <b>Materials</b>  | The grab bars are primarily constructed in 10-micrometer anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core.<br>The hinge arm is also made of soft SEBS.<br>Internal components and fasteners are primarily made of stainless steel.<br>The hinge barrel is capped with ABS polymer.   |



# PRESSALIT®

|                    |  |
|--------------------|--|
| <b>Product</b>     | PLUS fold down grab bar with integrated counter-balance, 33.5", right hand operated  |
| <b>Item number</b> | R370485277   |
| <b>Ergonomy</b>    | The oval shape gives an easy and comfortable surface to lean and push against. The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends. Soft and slip-resistant material that has a warm feel. |
| <b>Features</b>    | Folds against the wall.<br>The underside is shaped for the mounting of various accessories.  |
| <b>Warranty</b>    | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.  |



# PRESSALIT®

|                                  |   |
|----------------------------------|---|
| <b>Product</b>                   | PLUS fold down grab bar with integrated counter-balance, 33.5", right hand operated   |
| <b>Item number</b>               | R370485277  |
| <b>Cleaning</b>                  | <p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b><br/>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningssprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%<br/>Oxyplus-acid 2% (Diversey)<br/>WipeClean UniversalCleaning<br/>Chlorine Disinfection Wipes</p> <p><b>NON-DISINFECTANTS:</b><br/>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b><br/>Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p> |
| <b>Declaration of conformity</b> | This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.  |



# PRESSALIT<sup>®</sup>


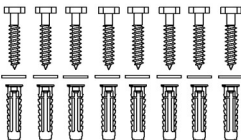

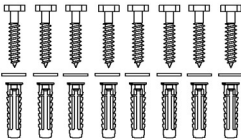
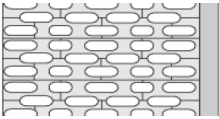
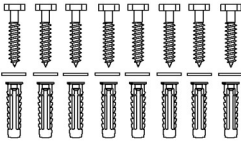
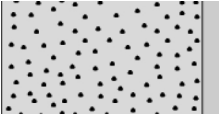
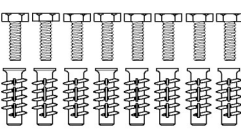
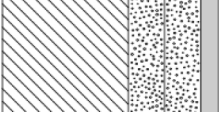
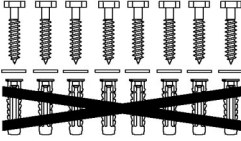
|                    |   |
|--------------------|---|
| <b>Product</b>     | PLUS fold down grab bar with integrated counter-balance, 33.5", right hand operated   |
| <b>Item number</b> | R370485277  |
|                    | <p><b>Intended use</b><br/>The fold down grab bar is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.</p> <p><b>Operating environment</b><br/>For indoor use in a wet environment, such as a bathroom or a washroom, in private homes, nursing homes and hospitals.<br/>For institutional use, the expected life time is 10 years.<br/><i>Life cycle test performed according to ISO 17966:2016.</i></p> <p>For maintenance, please refer to operation and maintenance manual, which is enclosed with the product, and which can also be found on the website under "Downloads".</p> |



PRESSALIT®

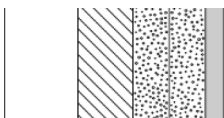
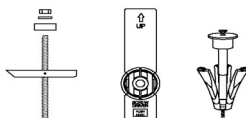

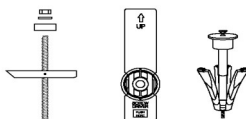
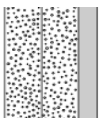
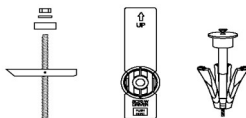
|                    |   |
|--------------------|---|
| <b>Product</b>     | PLUS fold down grab bar with integrated counter-balance, 33.5", right hand operated |
| <b>Item number</b> | R370485277  |

## Mounting in different wall types

|   |   |   |
|---|---|---|
|    | <b>Concrete &gt;B25</b><br>V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10.<br>To be ordered separately.  |    |
|   | <b>Solid Brick &gt;= Mz12</b><br>V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10.<br>To be ordered separately.  |   |
|  | <b>Solid Sand Lime Brick &gt;= Hlz12</b><br>V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10.<br>To be ordered separately.   |  |
|  | <b>Aerated Concrete P4</b><br>V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8.<br>To be ordered separately.  |  |
|  | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b><br>V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10.<br>NB! Plugs are not needed.<br>To be ordered separately. |  |



# PRESSALIT®

|   |   |
|---|---|
| <b>Product</b><br><b>Item number</b>  | PLUS fold down grab bar with integrated counter-balance, 33.5", right hand operated<br>R370485277   |
|    | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b></p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>            |
|    | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b></p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>  |
|  | <p><b>Drywall (2 x 12,5 mm) water-resistant</b></p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>    |

pressalit.com

Pressalit A/S  
 Pressalitvej 1  
 DK-8680 Ry  
 Tel.: +45 8788 8788  
 Fax: +45 8788 8789  
 E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®