

# PRESSALIT®

Product Item number	PLUS support arm with integrated counter-balance, 850 mm, left hand operated R370385000  VVS:615875200  NRF:6191084  RSK:8487442  EAN:5708590368092  HMI: 125881  FDA:ILS/D135866
Colour	White
Product description	All PLUS support arms come with an integrated counter-balance, which ensures a controlled movement between horizontal and vertical positions. Suitable for use with toilet pans projecting 700 mm or floor standing toilets. Replaces discontinued models R3175 og RF175.  PLUS support arm, counter-balanced. Projection 880 mm, ergonomically shaped to provide a clear sense of where the support arm ends. Left hand operated. Height adjustable by 240 mm without using tools. Detachable and foldable. Anthracite cover plate for vertical wall track.  Materials: The arms are constructed of 10-micrometre anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core Excl. mounting kit.
Adjustability	Height adjustable
Adjustment height	240 mm
Product capacity	Maximum load: 150 kg. When the support arm is loaded with 150 kg, each wall fastener in the vertical wall track is loaded with 50 kg.  PLUS support arms have been tested statically with 700 kg (close to the wall), 425 kg (in the middle) and 225 kg (the outermost point).



# PRESSALIT.

Product	PLUS support arm with integrated counter-balance, 850 mm, left hand operated
Item number	R370385000
Materials	The arms are primarily constructed in 10-micrometre anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core.  The hinge arm is also made of soft SEBS. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS polymer.
Ergonomy	The oval shape gives an easy and comfortable surface to lean and push against.  The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends.  Soft and slip-resistant material that has a warm feel.
Features	Folds against the wall.  The underside is shaped for the mounting of various accessories.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



## PRESSALIT®

	PLUS support arm with integrated counter-balance, 850 mm, left hand
Product	operated
Item number	R370385000
Cleaning	PLUS support arms. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.  The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.  DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Oxyplus-acid 2% (Diversey) WipeClean UniversalCleaning Chrolorine Disinfection Wipes  NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Millizid Citro  The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.  DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.
Declaration of conformity	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.



# PRESSALIT.

Product Item number	PLUS support arm with integrated counter-balance, 850 mm, left hand operated
	R370385000
	Intended use
	The support arm is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.
	Operating environment  For indoor use in a wet environment, such as a bathroom or a washroom, in private
	homes, nursing homes and hospitals.
	For institutional use, the expected life time is 10 years.  Life cycle test performed according to ISO 17966:2016.
	For maintenance, please refer to user guide, which is enclosed with the product, and which can also be found on the website under "Downloads".



### PRESSALIT.

PLUS support arm with integrated counter-balance, 850 mm, left hand operated

Item number

R370385000


### Installation in different wall types


a. V. O. (7 )
.C
D. V. D. O. O.
O. O. D. O. O. O.
a D : C : o

#### Concrete >B25

V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.

To be ordered separately.

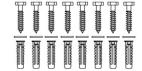

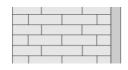


#### Solid Brick >= Mz12

V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.

To be ordered separately.




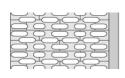


#### Solid Sand Lime Brick >= HIz12

V8610 - Mounting kit, 8 coach screws  $\emptyset 8$  x 70, plugs SX10.

To be ordered separately.

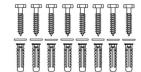


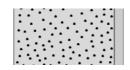


#### **Aerated Concrete P4**

V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8.

To be ordered separately.

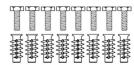

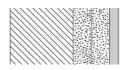


### Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.

NB! Plugs are not needed.

To be ordered separately.

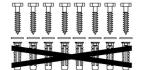



## PRESSALIT®

Product	PLUS support arm with integrated counter-balance, 850 mm, left hand operated	
Item number	R370385000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood  Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)  Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT.