




# PRESSALIT®

<b>Product</b> <b>Item number</b>	PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated R370385112
<b>Color</b>	VVS:615875212 NRF:6191087 RSK:8487443 EAN:5708590368139 HMI: 125881 FDA: ILS/D135866 Anthracite gray
<b>Product description</b>	 <p>All PLUS fold down grab bars come with an integrated counter-balance, which ensures a controlled movement between horizontal and vertical positions. Suitable for use with elongated toilet pans or floor standing toilets. <i>Replaces discontinued models R3175 og RF175.</i></p> <p><i>PLUS fold down grab bar, counter-balanced. Projection 34.7", ergonomically shaped to provide a clear sense of where the fold down grab bar ends. Left hand operated. Height adjustable by 9.5" without using tools. Detachable and foldable. Anthracite cover plate for vertical wall track.</i></p> <p><i>Materials: The arms are constructed of 10-micrometer anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core</i></p> <p><i>Excl. mounting kit.</i></p>
<b>Adjustability</b> <b>Adjustment height</b> <b>Product capacity</b>	Height adjustable 9,46 inch Maximum load: 330 lbs. When the fold down grab bar is loaded with 330 lbs, each wall fastener in the vertical wall track is loaded with 110 lbs.  <i>PLUS fold down grab bars have been tested statically with 1540 lbs (close to the wall), 935 lbs (in the middle) and 495 lbs (the outermost point).</i>

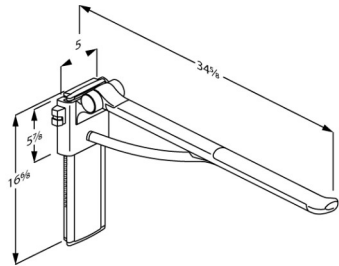

<b>Product</b> <b>Item number</b>	PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated R370385112
<b>Materials</b>  <b>Ergonomy</b>  <b>Features</b>	<p>The grab bars are primarily constructed in 10-micrometer anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core.</p> <p>The hinge arm is also made of soft SEBS.</p> <p>Internal components and fasteners are primarily made of stainless steel.</p> <p>The hinge barrel is capped with ABS polymer.</p> <p>The oval shape gives an easy and comfortable surface to lean and push against.</p> <p>The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends.</p> <p>Soft and slip-resistant material that has a warm feel.</p> <p>Folds against the wall.</p> <p>The underside is shaped for the mounting of various accessories.</p>
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



# PRESSALIT®

<b>Product</b>	PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated
<b>Item number</b>	R370385112
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Oxyplus-acid 2% (Diversey) WipeClean UniversalCleaning Chlorine Disinfection Wipes</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
<b>Declaration of conformity</b>	This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.




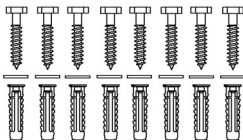

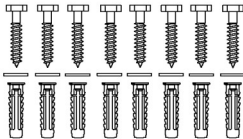
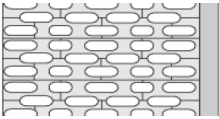
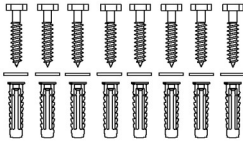
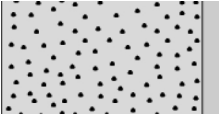
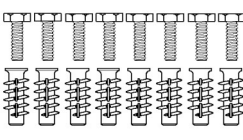
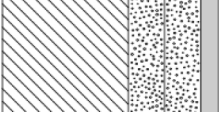
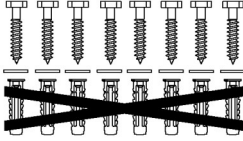
# PRESSALIT<sup>®</sup>

<b>Product</b>	PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated
<b>Item number</b>	R370385112
	<p><b>Intended use</b> The fold down grab bar is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.</p> <p><b>Operating environment</b> For indoor use in a wet environment, such as a bathroom or a washroom, in private homes, nursing homes and hospitals. For institutional use, the expected life time is 10 years. <i>Life cycle test performed according to ISO 17966:2016.</i></p> <p>For maintenance, please refer to operation and maintenance manual, which is enclosed with the product, and which can also be found on the website under "Downloads".</p>



<b>Product</b>	PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated
<b>Item number</b>	R370385112

## Mounting in different wall types

	<b>Concrete &gt;B25</b> V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Solid Brick &gt;= Mz12</b> V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Solid Sand Lime Brick &gt;= Hlz12</b> V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Aerated Concrete P4</b> V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b> V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



# PRESSALIT®

<b>Product</b> <b>Item number</b>	PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated R370385112
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b></p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b></p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>
	<p><b>Drywall (2 x 12,5 mm) water-resistant</b></p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>

pressalit.com

Pressalit A/S  
 Pressalitvej 1  
 DK-8680 Ry  
 Tel.: +45 8788 8788  
 Fax: +45 8788 8789  
 E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®