




# PRESSALIT®

|                                      |   |
|--------------------------------------|---|
| <b>Product</b><br><b>Item number</b> | PLUS support arm with integrated counter-balance, 700 mm, left hand operated<br>R372370112  |
| <b>Colour</b>                        | Anthracite grey   |
| <b>Product description</b>           | <p>           VVS:615877112<br/>           NRF:6191126<br/>           EAN:5708590368535<br/>           HMI: 138189         </p>  <p>           All PLUS support arms come with an integrated counter-balance, which ensures a controlled movement between horizontal and vertical positions.         </p> <p>           PLUS support arm, counter-balanced, 700 mm.<br/>           Height adjustable 150 mm. Foldable. Left handed.<br/>           Only for use with TL3/TL4 toilet lifter equipped with toilet pan projecting 540/550 mm.<br/>           Ergonomically shaped to provide a clear sense of where the support arm ends.<br/>           Materials: The arms are constructed of 10-micrometre anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core.<br/>           Excl. mounting kit.         </p> |
| <b>Adjustability</b>                 | Height adjustable   |
| <b>Adjustment height</b>             | 150 mm  |
| <b>Product capacity</b>              | Maximum load: 150 kg  |
| <b>Materials</b>                     | <p>           The arms are primarily constructed in 10-micrometre anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core.<br/>           The hinge arm is also made of soft SEBS.<br/>           Internal components and fasteners are primarily made of stainless steel.<br/>           The hinge barrel is capped with ABS polymer.         </p>   |
| <b>Ergonomy</b>                      | <p>           The oval shape gives an easy and comfortable surface to lean and push against.         </p> <p>           The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends.         </p> <p>           Soft and slip-resistant material that has a warm feel.         </p>   |

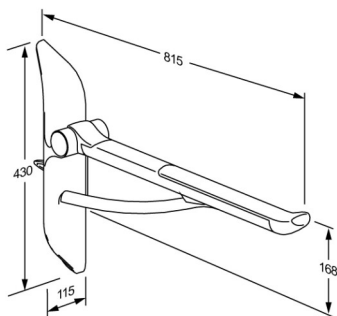

# PRESSALIT®

|                    |  |
|--------------------|--|
| <b>Product</b>     | PLUS support arm with integrated counter-balance, 700 mm, left hand operated   |
| <b>Item number</b> | R372370112   |
| <b>Features</b>    | The underside is shaped for the mounting of various accessories.   |
| <b>Warranty</b>    | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.  |
| <b>Cleaning</b>    | <p>PLUS support arms.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b></p> <p>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningssprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%<br/>Oxyplus-acid 2% (Diversey)<br/>WipeClean UniversalCleaning<br/>Chlorine Disinfection Wipes</p> <p><b>NON-DISINFECTANTS:</b></p> <p>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b></p> <p>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> |



# PRESSALIT®

|                                  |  |
|----------------------------------|--|
| <b>Product</b>                   | PLUS support arm with integrated counter-balance, 700 mm, left hand operated   |
| <b>Item number</b>               | R372370112   |
| <b>Declaration of conformity</b> | <p>CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.</p> <p><b>Intended use</b><br/>The support arm is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.</p> <p><b>Operating environment</b><br/>For indoor use in a wet environment, such as a bathroom or a washroom, in private homes, nursing homes and hospitals.<br/>For institutional use, the expected life time is 10 years.<br/><i>Life cycle test performed according to ISO 17966:2016.</i></p> <p>For maintenance, please refer to user guide, which is enclosed with the product, and which can also be found on the website under "Downloads".</p> |

[pressalit.com](http://pressalit.com)

## PRESSALIT®