

PRESSALIT.

Product	VALUE II support arm, fixed height		
Item number	R1109000		
	VVS:615891100		
	NRF:6191057		
	RSK:8487434		
	EAN:5708590367576		
	HMI: 123560		
Colour	White		
Product description	VALUE II support arm, fixed, foldable, projection 850 mm. The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.		
	VALUE II support arm, fixed, foldable, projection 850 mm. The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends. Materials: Rilsan-coated steel, pressure-cast aluminium, plastic. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.		
Adjustability	Fixed mounting		
Product capacity	Maximum load: 120 kg. When the support arm is loaded with 120 kg, each wall fastener is loaded with 242 kg.		
Materials	The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion.		
Ergonomy	Provides firm support when you are sitting, standing, or transferring to and from a toilet.		
	Good, broad support surface for the hand and arm. The ergonomic form gives a firm and comfortable grip.		
Features	The support arm can fold up when not in use or to make more space.		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.		



PRESSALIT.

Product	VALUE II support arm, fixed height		
Item number	R1109000		
Cleaning	VALUE support arms. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Klorilli NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 The product must be wiped down with a soft rag wrung out in clear water immediate after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.		
Declaration of conformity	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.		



PRESSALIT®

Product				
Item number				

VALUE II support arm, fixed height

R1109000

Installation in different wall types

<i>y</i> 1	
Concrete >B25 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
Solid Brick >= Mz12 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
Solid Sand Lime Brick >= HIz12 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
Aerated Concrete P4 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included.	

Plugs to be ordered separately.



PRESSALIT®

Product Item number	VALUE II support arm, fixed height R1109000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	!

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.

pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8789 Fax: +45 8788 8789

PRESSALIT.