

PRESSALIT.

T	
Product	PLUS folding seat, manually height adjustable
Item number	R5500000
	VVS:777531700
	NRF:7501062
	EAN:5708590328218
	HMI: 130162
Colour	White 424 370
Product description	PLUS folding seat. Manually height adjustable by 240 mm.
	The seat is foldable.
	For fitting to vertical PLUS wall track (not included). The folding seat is not suitable for use in the shower area.
	DLUS folding cost Manually height adjustable by 240 mm
	PLUS folding seat. Manually height adjustable by 240 mm. Materials: Aluminium, thermoset plastic, plastic.
	For fitting to vertical track (not included).
Adjustability	Height adjustable
Adjustment height	240 mm
Product capacity	Maximum user mass: 265 kg. When the seat is loaded with 265 kg, each wall fastener is loaded with 23 kg.
	The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.
Materials	The folding seat is made out of thermoset plastic, and the mounting and suspension
	bracket are made from 10 my anodised aluminium. The end caps are made from ABS plastic.
Ergonomy	Ergonomically shaped seat, which offers good and comfortable support. The seat is suitable for situations where additional seating is required and space is limited, - e.g. in kitchens, lifts and corridors or on the landings.
Features	The seat can be folded away when not in use, - or to free up space. Stays up of itself when folded away.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT.®

Product	PLUS folding seat, manually height adjustable
Item number	R5500000
Cleaning	Folding seat. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.
Declaration of conformity	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.



PRESSALIT®

Product PLUS folding seat, manually height adjustable

Item number R5500000

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com