




<p>Product Item number</p>	<p>SCT 2000 shower change table, coated mesh cover, fixed height R8425218000</p>
<p>Colour Product description</p>	<p>VVS:777543020 EAN:5708590372167 HMI: 130360 FDA:INT/D124320</p> <p>White</p>  <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>SCT 2000 shower change table, fixed height , foldable. Incl. safety rail and water collection tray with integrated outlet. Outer measurements 860 mm x 1800 mm. Materials: Powder-lacquered stainless steel, coated mesh cover and polystyrene. Excl. mounting kit.</p>
<p>Adjustability Product capacity</p>	<p>Fixed mounting</p> <p>Maximum user weight (MUM): 175 kg. Safe working load (SWL): 175 kg. When the table is loaded with 175 kg, each mounting bolt supports 52 kg <i>This product has been tested statically with a load of 262,5 kg as the ISO 17966 standard requires testing with safety factor 1.5.</i></p>
<p>Materials Ergonomy</p>	<p>Construction in powder-laquered stainless steel, mesh cover in PVC-coated polyester (for extra durability) and water collection tray in PVC-U.</p> <p>The 2000 series shower and changing tables has a low start position, to make transfers to the bench easier. The washable, perforated mesh cover lets the water pass through to the water collection tray underneath. The flexibility of the shower and changing table provides for optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p>

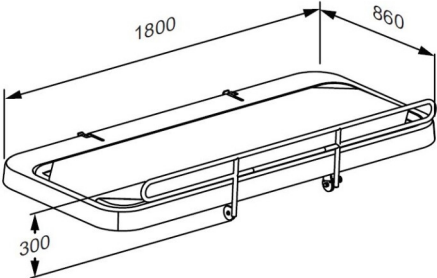

Product Item number	SCT 2000 shower change table, coated mesh cover, fixed height R8425218000
Features	<p>A water collection tray with integrated water outlet prevents water from splashing on carers or the floor. The flexible hose attached to the centre mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall. The integrated gas cylinders enable the table to be folded out of the way when not in use to maximise floor space in front of the unit.</p> <p>Electric with wired hand control – the red buttons clearly indicate “Up” and “Down”. The wired hand control comes with a holder for fitting to the wall.</p> <p>The stretcher of the table is very easy to clean, and the stretcher cover can be unzipped for washing or easy replacement.</p>
Warranty	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.</p>
Cleaning	<p>SCT 1000 & 2000.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>



PRESSALIT®

Product	SCT 2000 shower change table, coated mesh cover, fixed height
Item number	R8425218000
Declaration of conformity	CE UK MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices. For the UK: - Medical Device Regulation (MDR) 2002 on medical devices.









Product	SCT 2000 shower change table, coated mesh cover, fixed height
Item number	R8425218000

Installation in different wall types

	<p>Concrete >B25 V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12. To be ordered separately.</p>	
	<p>Solid Brick >= Mz12 V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12. To be ordered separately.</p>	
	<p>Solid Sand Lime Brick >= Hlz12 V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12. To be ordered separately.</p>	
	<p>Aerated Concrete P4 V8658 - Mounting kit, 6 coach screws Ø10 x 70 - plugs GB14. To be ordered separately.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12. NB! Plugs are not needed. To be ordered separately.</p>	

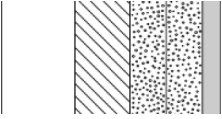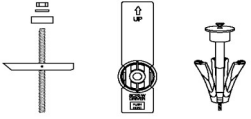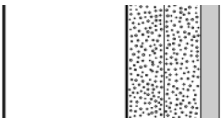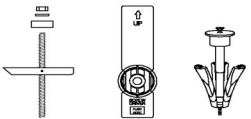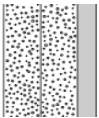

Product Item number	SCT 2000 shower change table, coated mesh cover, fixed height R8425218000
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

It is the responsibility of the installer to ensure that suitable wall fasteners are used.