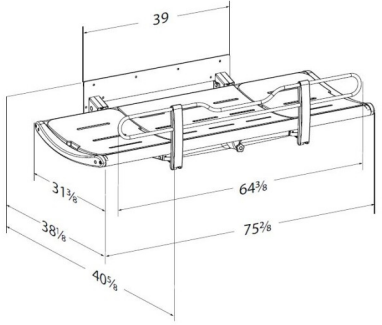




# PRESSALIT®

<b>Product</b> <b>Item number</b>	SCT 3000 shower change table, fixed height R8435318299
<b>Color</b>	VVS: 777543198 EAN: 5708590374031 HMI: 130171 FDA: INT/D124320  Sapphire blue (RAL 5003)
<b>Product description</b>	 <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>The SCT 3000 series has a stylish and award-winning design focused on comfort and ease of use.</p> <p>SCT 3000 shower change table, fixed height.  Foldable. Incl. safety rail, flexible hose (Ø1.26") and water collection tray with integrated outlet.  Surface length 70.9", outer measurements 40.6" x 75.2".  Materials: Powder-lacquered stainless steel, aluminium, polyurethane foam and polystyrene.  Excl. mounting kit.</p>
<b>Adjustability</b> <b>Product capacity</b>	Fixed mounting  Maximum user weight (MUM): 441 lbs. Maximum load (SWL): 463 lbs. When the table is loaded with 441 lbs, each mounting bolt supports 132 lbs. <i>This product has been tested statically with a load of 694 lbs, as the ISO 17966 standard requires testing with a safety factor 1.5.</i>
<b>Materials</b>	Aluminium, polyurethane foam, plastics, powder-coated steel.



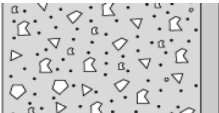
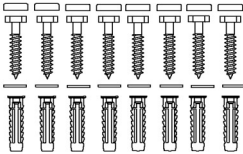

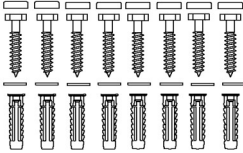
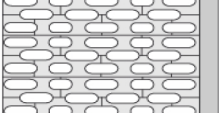
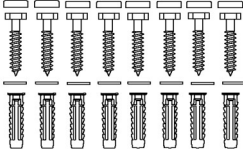
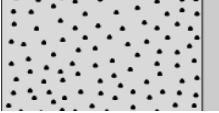
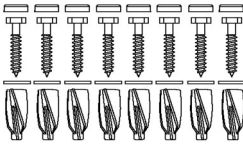
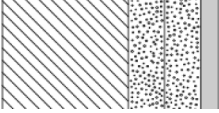
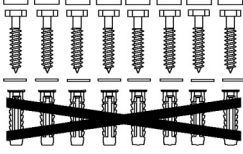
<b>Product</b>	SCT 3000 shower change table, fixed height
<b>Item number</b>	R8435318299
<b>Ergonomy</b>	<ul style="list-style-type: none"><li>• The ergonomically designed safety rails have a practical lower central section, which provides better access and work environment for the carer while transferring, showering, drying, changing and dressing the user.</li><li>• One hand operated safety rails allows the carer to maintain eye contact as well as physical contact with the user during the whole process.</li><li>• Easy to clean, smooth contact surfaces which do not collect dirt and bacteria.</li></ul>
<b>Features</b>	<ul style="list-style-type: none"><li>• A water collection tray with integrated water outlet prevents water from splashing on caregiver or the floor. The flexible hose (Ø1.26") attached to the center mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall.</li><li>• Adjustable neck support which can be moved to either end of the 51.2" changing tables.</li><li>• The slightly curved shape makes the table comfortable and safe to lie on •The integrated pneumatic cylinders enable the table to be folded out of the way when not in use to maximize floor space.</li></ul>
<b>Warranty</b>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.</p>
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: CaviCide Accel PREvention Wipes Lingettes</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
<b>Declaration of conformity</b>	<p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>



PRESSALIT®

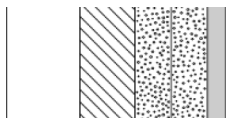
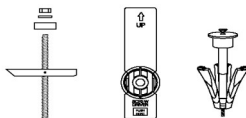
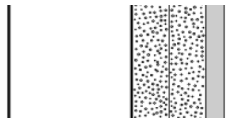
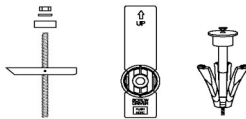
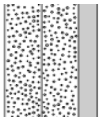

<b>Product</b>	SCT 3000 shower change table, fixed height
<b>Item number</b>	R8435318299

## Mounting in different wall types

	<b>Concrete &gt;B25</b> V8613 - Mounting kit, 8 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
	<b>Solid Brick &gt;= Mz12</b> V8613 - Mounting kit, 8 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
	<b>Solid Sand Lime Brick &gt;= Hlz12</b> V8613 - Mounting kit, 8 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
	<b>Aerated Concrete P4</b> V8663 - Mounting kit, 8 lag bolts Ø10 x 70, plugs GB14, with grey covers. To be ordered separately.	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b> V8613 - Mounting kit, 8 lag bolts Ø10 x 70, plugs SX12, with grey covers. NB! Plugs are not needed. To be ordered separately.	



# PRESSALIT®

<b>Product</b> <b>Item number</b>	SCT 3000 shower change table, fixed height R8435318299
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b> Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

[pressalit.com](http://pressalit.com)