


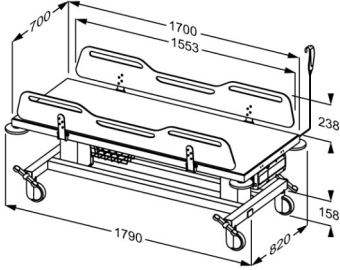

<p>Product Item number</p>	<p>MCT 2 change trolley, electrically height adjustable R9412717000</p>
<p>Colour Product description</p>	<p>EAN:5708590373164</p>  <p>White</p> <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>MCT 2 change trolley, 820 x 1790 mm, two fold-down sides. Electrically height adjustable (battery operated) with wired remote control. 400 mm lifting range, from 550 to 950 mm. Equipped with central lock system, wire basket for convenient storage, bumpers on all four corners as well as battery and charger. Lifting columns and wired remote control IPX6, battery and charger IPX5. Mattress to be ordered separately.</p>
<p>Adjustability Adjustment height Product capacity Speed upwards</p>	<p>Height adjustable</p> <p>400 mm</p> <p>Maximum user weight (MUM): 200 kg. Maximum load (SWL): 250 kg. (user weight + water + accessories). <i>This product has been tested statically with a load of 625 kg as the IEC 60601-1 standard requires testing with safety factor 2.5.</i></p> <p>Speed up mm: 19 mm/s Speed down mm: 19 mm/s</p>
<p>Materials</p>	<p>Aluminium, plastics, compact laminate and powder-lacquered stainless steel.</p>



Product	MCT 2 change trolley, electrically height adjustable
Item number	R9412717000
Ergonomy	<ul style="list-style-type: none">• The ergonomically designed safety rails have a practical lower central section, which provides better access and work environment for the carer while transferring, changing and dressing the user.• One hand operated safety rails allows the carer to maintain eye contact as well as physical contact with the user during the whole process, the foldable safety rails also provide easy access from all four sides of the table.• The cut-out grips in the safety rails enable users to assist when being turned, which motivates physical training.• A storage drawer enables the carer to store supplies and place what is needed for the shower immediately at hand in advance.• Easy to clean, smooth contact surfaces which do not collect dirt and bacteria.
Features	<ul style="list-style-type: none">• The trolley can be moved from room to room for easy transport and multiple users. Two lifting columns are used to create an aesthetically pleasing, hygienically optimized, physically and visually stable construction.• Intuitive, easy-to-use drop down rails with one hand operation. • The centralized brake system allows the carer to simultaneously lock all four wheels by stepping on any single activator, ensuring better safety and ease of use in any position.• All locking and release activators are activated by stepping down on them from above to ensure ergonomic interaction, and to prevent foot and toe injuries. Non-slip surface and pattern on all brakes prevent slipping even in wet conditions.• A rotating bumper on each of the trolley's corners protects surrounding walls and the product itself when in contact and during relocation.• Powered height adjustment with battery and charger. The wall-hung charger for the battery can be mounted in any room, incl. outside the bathroom if so desired. When the battery is running low, a beep will sound each time the wired remote control is activated.• Battery charging time 3-4 hours. • Turning diameter: 1935 mm.
Mains connection (V)	230
Actuator (motor)	BL1 (Linak) IPX6
Control box	COL50 (Linak) IPX6
Wired remote control	HB30 (Linak) IPX6
Power consumption (maximum load)	Power consumption by maximum load, watt: 30 W Power consumption by standby: 0,5 W



Product	MCT 2 change trolley, electrically height adjustable
Item number	R9412717000
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.
Cleaning	<p>MSCT 1, MCT 2+3</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
Declaration of conformity	<p>CE UK MDR-MASK-RoHS This product is in compliance with all relevant requirements of:</p> <ul style="list-style-type: none">- Regulation (EU) 2017/745 on medical devices- Council Directive 2006/42/EC on machinery- Council Directive 2011/65/EC on RoHS. For the UK:- Medical Device Regulation (MDR) 2002 on medical devices.- Supply of Machinery (Safety) Regulations 2008- Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012.



PRESSALIT®

Product	MCT 2 change trolley, electrically height adjustable
Item number	R9412717000

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®