

Product	PLUS shower seat 450 with aperture, electrically height adjustable		
ltem number	R7835112		
	EAN: 5708590374437 FDA: INN/D135865		
Color	Anthracite gray		
Product description	PLUS shower seats are offered in a number of colours. They are available as both fixed height, electrically adjustable and manually adjustable, with and without aperture, and with an option of back and armrests. They are suitable for wall-mounting or for fitting to Pressalit PLUS wall tracks.		
	Powered PLUS shower seat 450 with aperture, with back- and armrests. Electrically height adjustable by 19.7", the armrests are independently height adjustable. The seat and the armrests are foldable. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile. The non-slip gripping surface of the armrests are made of soft SEBS. Excl. mounting kit. <i>Replaces R7471</i>		
Adjustability	Height adjustable		
Adjustment height	19,7 inch		
Product capacity	Maximum user mass (seat): 330 lbs. Maximum load for armrests is 55 kg. When the seat is loaded with 330 lbs, each wall fastener is loaded with 134 lbs. The seat has been tested statically with safety factor 2.5, which is a requirement according to the applied assistive products standards.		
Speed up	Speed up: 0,39 inch/s Speed down: 0,39 inch/s		
Noise level	48 dB		



Product Item number	PLUS shower seat 450 with aperture, electrically height adjustable R7835112	
Materials	The shower seat is made of polyurethane (PU) foam with a plywood core and this is covered with a moulded coated surface that is protected using a clear PU lacquer. The backrest is made of polyurethane (PU) foam protected using a clear PU lacquer The arms are primarily constructed of 3/8" anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and inforporating a glass-reinforced nylon core. The hinge arm is also made of soft SEBS. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS	
Ergonomy	Ergonomically designed seat - contact surface are 17.7" x 17.2". Soft and slip- resistant material that has a warm feel. The oval shape of the armrests gives an ea and comfortable surface to lean on and push against.	
Features	Powered raising or lowering at the push of a button - the seat can easily be adjusted upwards or downwards to a good working height for the caregiver while the user sits on it. The seat, backrest and armrests can be adjusted simultaneously thanks to a safe and easy click system. The armrests can also be adjusted independently of the seat and backrest. The seat and armrests can fold up when not in use or to maximize floor space. Wired hand control comes with brackets for mounting on the wall.	
Mains connection (V)	120	
Actuator (motor)	LA40 (Linak) 6000N, IPX6	
Control box	SMPS14 (Linak) IPX6	
Wired remote control	HB3 (Linak) IP66	
Power consumption (maximum load)	Power consumption by maximum load: 60 W Power consumption by standby: 0,5 W	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	



Product	PLUS shower seat 450 with aperture, electrically height adjustable R7835112		
ltem number			
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.		
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.		
	DISINFECTANTS: Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)		
	NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)		
	Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anyth other than soap and water.		
	The product must be wiped down with a soft rag wrung out in clear water immedia after cleaning, as some materials may otherwise be discolored and stained, and t surface coating may deteriorate.		
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from w tracks and other aluminum parts using a solution of three parts of 30% acetic acid seven parts tap water. When the aluminum parts have been washed down with th solution, the parts should be wiped over with a damp cloth wrung out in plain wate		
Declaration of conformity	This product is solely intended for the US and Canada, and has no EU declaration conformity.		



Product

Item number

PLUS shower seat 450 with aperture, electrically height adjustable R7835112

Mounting in different wall types

 0	
Concrete >B25 V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
Solid Brick >= Mz12 V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
Solid Sand Lime Brick >= HIz12 V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
Aerated Concrete P4 V8653 - Mounting kit, 4 bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



Product Item number	PLUS shower seat 450 with aperture, electrically height adjustable R7835112		
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	Drywall (2 x 12,5 mm) water-resistant Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		

pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com