

PRESSALIT.

Product	SELECT TL3 toilet lifter, manual, for wall outlet		
Item number	R9023000		
	VVS:617989000		
	NRF:6191182		
	EAN:5708590372839		
	HMI: 138237		
Colour	White		
Product description	SELECT TL3 and TL4 toilet lifters are developed with a clear focus on hygiene.		
	SELECT TL3 toilet lifter, for wall outlet. Incl. cistern with dual flushing system preset at 3/6 litres (can be changed to 2/4 litres). Manually height adjustable 200 mm with crank handle. Toilet pan, toilet seat and support arms to be ordered separately. Materials: Stainless steel and PMMA/ABS. Excl. wall mounting kit.		
Adjustability	Height adjustable		
Adjustment height	200 mm		
Product capacity	Maximum user weight: 200 kg. Maximum load: 235 kg. (user weight + accessories). When the toilet lifter is loaded with 235 kg, each wall fastener is loaded with 65 kg. The toilet lifter has been tested statically with 600 kg, which is a requirement according to the applied assistive products standards. Please note that most toilet pans compatible with this lifter have been tested up to 400 kg.		
Materials	The toilet lifter is constructed of stainless steel, the cover fairing is made of PMMA/ABS plastics.		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.		



PRESSALIT.

Product	SELECT TL3 toilet lifter, manual, for wall outlet	
Item number	R9023000	
Cleaning	SELECT TL3, TL4 toilet lifter. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the	
Declaration of conformity	surface coating may deteriorate. CE UK MDR-MASK This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices - Council Directive 2006/42/EC on machinery For the UK: - Medical Device Regulation (MDR) 2002 on medical devices Supply of Machinery (Safety) Regulations 2008	



PRESSALIT®

Prod	luct
ltom	numh

SELECT TL3 toilet lifter, manual, for wall outlet

R9023000

Installation in different wall types

Concrete >B25 V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with grey screw covers. To be ordered separately.	
Solid Brick >= Mz12 V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with grey screw covers. To be ordered separately.	
Solid Sand Lime Brick >= HIz12 V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with grey screw covers. To be ordered separately.	
Aerated Concrete P4 V8655 - Mounting kit, 4 coach screws Ø10 x 70, plugs GB14, with grey screw covers. To be ordered separately.	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with white screw covers. NB! Plugs are not needed. To be ordered separately.	



PRESSALIT®

Product Item number	SELECT TL3 toilet lifter, manual, for wall outlet R9023000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>^!</u>

pressalit.com