

Product	CT 5000 change table, fixed height		
Item number	R8455512000		
	VVS:777540820		
	EAN: 5708590376585	700	
	HMI: 138135		
	FDA: INT/D124320		
Colour	White		
Product description	Pressalit offers a variety of mobile and wall-hung, height-adjustable and f shower and changing tables of high quality. They are designed to provide platform for showering and changing adults and children with reduced mo These tables can be used in schools, institutions, hospitals and private ho are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions carer as well as a high level of comfort and safety for the user.	e a stable obility. omes and	
	CT 5000 change table, lying surface in compact laminate, 700 x 1200 mr With safety rail and body belt, foldable, fixed height, wall-mounted.	n , white.	
Adjustability	Fixed mounting		
Adjustment height	0 mm		
Product capacity	Maximum user weight (MUM): 150 kg. Safe working load (SWL): 150 kg. When the table is loaded with 150 kg, each mounting bolt supports 52 kg		
Product capacity	Safe working load (SWL): 150 kg.		
Product capacity Materials	Safe working load (SWL): 150 kg. When the table is loaded with 150 kg, each mounting bolt supports 52 kg This product has been tested statically with a load of 225 kg as the ISO 1		



Product	CT 5000 change table, fixed height		
Item number	R8455512000		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date.		
Cleaning	CT 4000, 4100, 5000 The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.		
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREVention Wipes Lingettes		
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediar after cleaning, as some materials may otherwise be discolored and stained, and th surface coating may deteriorate.		
Declaration of conformity	CE UK MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices. For the UK: - Medical Device Regulation (MDR) 2002 on medical devices.		



Product

Item number

CT 5000 change table, fixed height R8455512000

Installation in different wall types

Concrete >B25 V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	
Solid Brick >= Mz12 V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	
Solid Sand Lime Brick >= HIz12 V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	
Aerated Concrete P4 V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	



Product Item number	CT 5000 change table, fixed height R8455512000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>!</u>
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>.</u>
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.

pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com

