


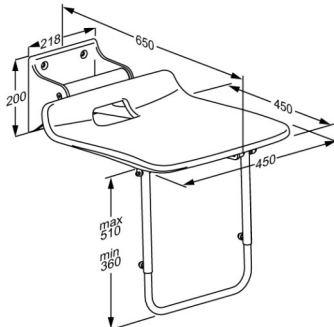

<b>Product</b> <b>Item number</b>	VALUE IV folding shower seat with supporting leg, fixed height R1622000112
<b>Colour</b>	VVS:777525412 NRF:6314098 RSK:8162782 EAN:5708590377728 HMI: 124486 DB: 2510921 FDA:INN/D135865 Anthracite grey
<b>Product description</b>	 <p>VALUE IV folding shower seat and backrest create a simple solution for users who need to sit down while showering.</p> <p>VALUE IV shower seat. Fixed height. The seat is foldable. Materials: Diecast aluminium, Rilsan-coated stainless steel and vacuum-moulded polyethylene (PE). Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p>
<b>Adjustability</b> <b>Product capacity</b>	Fixed mounting  Maximum user mass: 135 kg. When the seat is loaded with 135 kg, each wall fastener is loaded with 125 kg.  <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
<b>Materials</b>  <b>Ergonomy</b>  <b>Features</b>	<p>The wall profile is made of diecast aluminium, and the frame and the leg is made of Rilsan-coated stainless steel. Rilsan provides good protection against corrosion and is comfortable to touch and easy to clean. The seat is made of vacuum-moulded polyethylene (PE).</p> <p>Ergonomic seat with a comfortable surface.</p> <p>The seat can fold up when not in use or to make more space.</p> <p>Leg for extra support.</p>



# PRESSALIT®

<b>Product</b>	VALUE IV folding shower seat with supporting leg, fixed height
<b>Item number</b>	R1622000112
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
<b>Cleaning</b>	<p>VALUE shower seat</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b></p> <p>Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean)</p> <p><b>NON-DISINFECTANTS:</b></p> <p>Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
<b>Declaration of conformity</b>	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.



# PRESSALIT®

<b>Product</b>	VALUE IV folding shower seat with supporting leg, fixed height
<b>Item number</b>	R1622000112

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®