


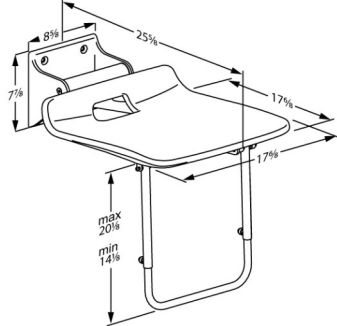

|  |  |
|--|--|
| <b>Product</b><br><b>Item number</b>                           | VALUE IV folding shower seat with supporting leg, fixed height<br>R1622000112  |
| <b>Color</b>   | VVS:777525412<br>NRF:6314098<br>RSK:8162782<br>EAN:5708590377728<br>HMI: 124486<br>Anthracite gray   |
| <b>Product description</b>                                     |  <p>VALUE IV folding shower seat and backrest create a simple solution for users who need to sit down while showering.</p> <p>Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately. VALUE IV shower seat. Fixed height. The seat is foldable.<br/>Materials: Diecast aluminium, Rilsan-coated stainless steel and vacuum-moulded polyethylene (PE).<br/>Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p> |
| <b>Adjustability</b><br><b>Product capacity</b>                | Fixed mounting<br><br>Maximum user mass: 275 lbs.<br>When the seat is loaded with 275 lbs, each wall fastener is loaded with 209 lbs.<br><br><i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>  |
| <b>Materials</b><br><br><b>Ergonomy</b><br><br><b>Features</b> | <p>The wall profile is made of diecast aluminium, and the frame and the leg is made of Rilsan-coated stainless steel. Rilsan provides good protection against corrosion and is comfortable to touch and easy to clean. The seat is made of vacuum-moulded polyethylene (PE).</p> <p>Ergonomic seat with a comfortable surface.</p> <p>The seat can fold up when not in use or to create more space.<br/>Leg for extra support.</p>   |
| <b>Warranty</b>  | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.  |



# PRESSALIT®

|                                  |  |
|----------------------------------|--|
| <b>Product</b>                   | VALUE IV folding shower seat with supporting leg, fixed height   |
| <b>Item number</b>               | R1622000112  |
| <b>Cleaning</b>                  | <p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b></p> <p>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningssprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%<br/>WipeClean UniversalCleaning<br/>Chlorine Disinfection Wipes<br/>Klorilli (Kiilto Clean)</p> <p><b>NON-DISINFECTANTS:</b></p> <p>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> |
| <b>Declaration of conformity</b> | <p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>  |



# PRESSALIT®

|                    |  |
|--------------------|--|
| <b>Product</b>     | VALUE IV folding shower seat with supporting leg, fixed height |
| <b>Item number</b> | R1622000112  |

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®