




PRESSALIT®

Product	MCT 3 change trolley, electrically height adjustable
Item number	R9652717000
Colour	White
Product description	<p>EAN:5708590389950</p>  <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>MCT 3 change trolley, 700 x 1700 mm, two fold-down sides. Electrically height adjustable with wired remote control. 500 mm lifting range, from 450 to 950 mm.</p> <p>Mattress to be ordered separately.</p>
Adjustability	Height adjustable
Adjustment height	500 mm
Product capacity	<p>Maximum user weight (MUM): 200 kg. Maximum load (SWL): 205 kg. <i>This product has been tested statically with a load of 512,5 kg as the IEC 60601-1 standard requires testing with safety factor 2.5.</i></p>
Speed upwards	<p>Speed up mm: 17 mm/s Speed down mm: 17 mm/s</p>
Noise level	50 dB
Materials	Plastics, compact laminate and powder-lacquered steel.

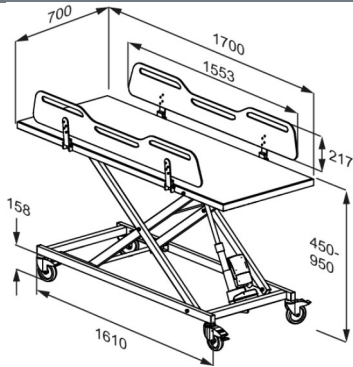

Product	MCT 3 change trolley, electrically height adjustable
Item number	R9652717000
Ergonomy	<ul style="list-style-type: none"> • The ergonomically designed safety rails have a practical lower central section, which provides better access and work environment for the carer while transferring, changing and dressing the user. • One hand operated safety rails allows the carer to maintain eye contact as well as physical contact with the user during the whole process, the foldable safety rails also provide easy access from all four sides of the table. • The cut-out grips in the safety rails enable users to assist when being turned, which motivates physical training. • Easy to clean, smooth contact surfaces which do not collect dirt and bacteria.
Features	<ul style="list-style-type: none"> • The trolley can be moved from room to room for easy transport and multiple users. • Intuitive, easy-to-use drop down rails with one hand operation.
Mains connection (V)	230
Actuator (motor)	LA40 (Linak) IPX6
Control box	CA30 (Linak) IPX6
Wired remote control	HB30 (Linak) IPX6
Power consumption (maximum load)	Power consumption by maximum load, watt: 330 W Power consumption by standby: 0,5 W
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.



PRESSALIT®

Product	MCT 3 change trolley, electrically height adjustable
Item number	R9652717000
Cleaning	<p>MSCT 1, MCT 2+3</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS:</p> <p>Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes</p> <p>NON-DISINFECTANTS:</p> <p>Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
Declaration of conformity	<p>UK MDR-MASK-RoHS This product is in compliance with all relevant requirements of:</p> <ul style="list-style-type: none">- Medical Device Regulation (MDR) 2002 on medical devices.- Supply of Machinery (Safety) Regulations 2008- Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012.



PRESSALIT®

Product	MCT 3 change trolley, electrically height adjustable
Item number	R9652717000

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®